


Bluetooth®

Pairing

Quick Guide

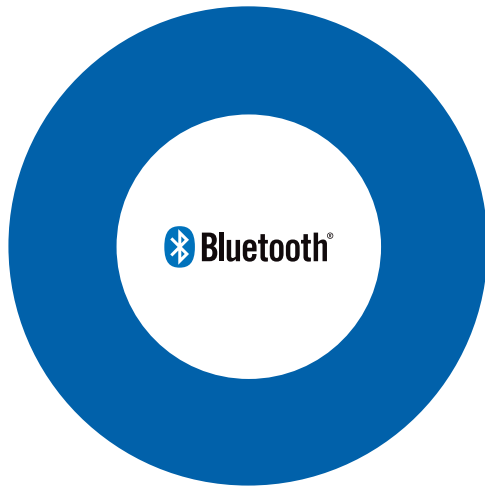

GLUCOCARD®

Read Me First

- Visit datamanagement@arkrayusa.com to download the compatible data management mobile app.
- Some smartphones especially those that are not tested or approved by the manufacturer, may be incompatible with your meter.
- Descriptions are based on the default settings of the meter and smartphones.
- Please read the meter's user manual before using the device to ensure safe and proper use.

Tips for Using Bluetooth®

- To avoid problems when connecting your meter to a smartphone, place the devices close to each other.
- Ensure that your meter and the other Bluetooth® device are within the connection range 32 feet (10 meters).
- Ensure that there are no obstacles between your meter and smartphone. If there are obstacles between the devices, the operating distance may be reduced.

Pairing your Meter with the Compatible Mobile App

1. Open the app on your mobile device (such as a smartphone).
2. Ensure that your meter and the mobile device are within the connection range of 32 feet (10 meters).
3. Initiate your meter's Bluetooth® pairing mode by one of the following two methods:
 - a. Take a blood glucose reading. When you are finished taking the reading, remove the strip which will automatically place your meter into Bluetooth® pairing mode...or,
 - b. Press and hold down your meter's right arrow button for 3 seconds. This will place your meter into Bluetooth® pairing mode.
4. The Bluetooth® icon will blink, indicating your meter is in Bluetooth® pairing mode.
5. Follow the pairing instructions of the app on the mobile device that you are pairing with your meter.



The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by the manufacturer, is under license.

ARKRAY Technical Customer Service: **800.566.8558**,
24 hours a day, 7 days a week.

In case of emergency, please contact your healthcare professional or emergency medical response.

Distributed By:

ARKRAY USA
8260 NW 27th St, Suite 403
Miami, FL 33122 USA
U.S. Support
800.566.8558
www.arkrayusa.com

©ARKRAY USA, Inc. All Rights Reserved.
P/N 5526-03 Rev. 06/25
PGUAB-0000015 Rev3 06/2025